



Weck  
**Hem-o-lok**  
 Polymer Locking Ligation System

**Secure-locking technology**

The Weck Hem-o-lok Polymer Ligation System delivers unparalleled ligation security compared to leading competitors,\* with the Weck Hem-o-lok Polymer Ligation Clip's integrated ridges and proprietary distal locking mechanism.

**Polymer flexibility**





The Hem-o-lok Clip's non-absorbable polymer composition and flexible hinge allow it to pass through a smaller port and provide a larger distal opening than comparable metal clips. As a result, Hem-o-lok Clips can ligate up to a 10 mm structure through a 5 mm port, and up to a 16 mm structure through a 10 mm port.

**Cold ligation**

Since it's a cold ligation system, with the Hem-o-lok Clip there's no chance of thermal spread to vital structures – a common concern with energy-based ligation solutions.

**Size ranges to fit your needs**

Hem-o-lok Clips come in a range of sizes indicated to ligate structures from 2 mm to 16 mm.

-  M: 2 mm – 7 mm
-  L: 5 mm – 13 mm
-  M/L: 3 mm – 10 mm
-  XL: 7 mm – 16 mm

**Benefits**



**The Clinician**

The Hem-o-lok Clip's integrated ridges and distal locking mechanism provide unparalleled clip security compared to leading competitors.\*



**Your Institution**


Cost-savings opportunity compared to leading competitors.\*\*



**The Patient**

Flexible clip hinge enables ligation of larger tissue structures through smaller ports, helping to minimise patient trauma.

## Weck Hem-o-lok Polymer Ligation Clip Features

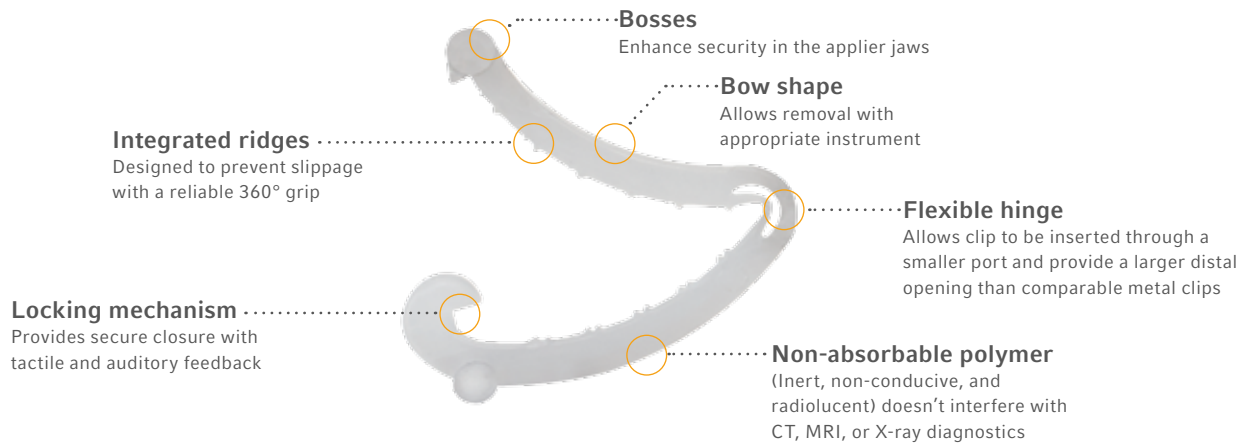

## Weck Hem-o-lok Polymer Locking Ligation System Ordering Information

CLIPS	OPEN MANUAL LOAD APPLIERS/REMOVERS			ENDOSCOPIC MANUAL LOAD APPLIERS/REMOVERS			AUTOMATIC ENDOSCOPIC APPLIERS	
<b>544220</b> Hem-o-lok Medium Polymer Clips 6 clips/cartridge 14 cartridges/box sterile	544113	20 cm	curved					
	544115	27 cm	curved					
	544114	27 cm	right-angle (70°)					
	544123	27 cm	remover (for M, ML, L)					
<b>544230</b> Hem-o-lok Medium-Large Polymer Clips 6 clips/cartridge 14 cartridges/box sterile	544170	20 cm	curved	544965T	34 cm, 5 mm	applier TAP	AE05ML	35 cm, 5 mm
	544171	27 cm	curved	544965T20	34 cm, 5 mm	applier TAP 20°	15 ML clips	
	544172	27 cm	right-angle (70°)	544965L	45 cm, 5 mm	applier long	3 appliers/box	
	544123	27 cm	remover (for M, ML, L)	544121T	32 cm, 5 mm	remover TAP (for ML, L)	sterile	
				544121L	45 cm, 5 mm	remover long (for ML, L)	disposable	
<b>544240</b> Hem-o-lok Large Polymer Clips 6 clips/cartridge 14 cartridges/box sterile	544180	20 cm	curved	544995T	34 cm, 10 mm	applier TAP		
	544182	25 cm	right-angle, narrow shaft	544995T20	34 cm, 10 mm	applier TAP 20°		
	544181	27 cm	curved	544995L	45 cm, 10 mm	applier long		
	544179	27 cm	right-angle (70°)	544121T	32 cm, 5 mm	remover TAP (for ML, L)		
	544123	27 cm	remover (for M, ML, L)	544121L	45 cm, 5 mm	remover long (for ML, L)		
	544124	27 cm	remover (L, XL)	544130T	33 cm, 10 mm	remover TAP (for L, XL)		
				544130T20	33 cm, 10 mm	remover TAP 20° (for L, XL)		
				544130L	45 cm, 10 mm	remover long (for L, XL)		
<b>544250</b> Hem-o-lok Extra-Large Polymer Clips 6 clips/cartridge 14 cartridges/box sterile	544191	27 cm	curved	544990T	34 cm, 10 mm	applier TAP		
	544192	27 cm	right-angle (70°)	544990T20	34 cm, 10 mm	applier TAP 20°		
	544124	27 cm	remover (L, XL)	544990L	45 cm, 10 mm	applier long		
				544130T	33 cm, 10 mm	remover TAP (for L, XL)		
				544130T20	33 cm, 10 mm	remover TAP 20° (for L, XL)		
				544130L	45 cm, 10 mm	remover long (for L, XL)		
<b>Storage Tray</b>				544500	capacity: 3 endoscopic 32 cm appliers			

TAP = take-a-part

## For further information, contact your Teleflex surgical sales professional.

To place an order, contact customer service at +353 (0)9 06 46 08 00.

\*Data on file (2013 internal study), Teleflex Incorporated, Report #D001591. Based on ex-vivo testing of three common clip failure modes conducted on porcine carotids, sample size = 33, p<0.05. Clinical performance cannot be extrapolated from the data. Testing pressures range beyond physiological pressures. The Weck Hem-o-lok medium/large clip was compared to Ethicon LIGAMAX, Ethicon 10 mm LIGACLIP, and Covidien Endo Clip III. The Weck Hem-o-lok large clip was compared to Ethicon 12 mm LIGACLIP and Covidien Endo Clip.\*\*Automatic metal ligation competitors, endoscopic internal stapling devices and advanced energy systems.

Hem-o-lok Ligating Clips are contraindicated for use as a fallopian contraceptive tubal occlusion device and contraindicated for use in ligating the renal artery during laparoscopic donor nephrectomies. Ethicon, LIGACLIP, and LIGAMAX 5 are trademarks or registered trademarks of Johnson & Johnson Corporation. Covidien and Endo Clip are registered trademarks of Covidien LP. Teleflex, the Teleflex logo, Hem-o-lok, and Weck are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Information in this document is not a substitute for the product Instructions for Use. The products in this document may not be available in all countries. Please contact your local representative. All data current at time of printing (05/2017). Subject to technical changes without further notice. © 2017 Teleflex Incorporated. All rights reserved. 94 30 55-00 00 01 REV B 05 17 02

**Teleflex**

Distributed by:  
Teleflex Headquarters International, Ireland · Teleflex Medical Europe Ltd. · IDA Business & Technology Park  
Dublin Road · Athlone · Co Westmeath · Tel. +353 (0)9 06 46 08 00 · Fax +353 (0)14 37 07 73 · orders.int@teleflex.com  
United Kingdom Tel. +44 (0)14 94 53 27 61 · info.uk@teleflex.com  
South Africa Tel. +27 (0)11 807 4887 · assist.africa@teleflex.com