



MEMOBAG

A bag with memory

AT A GLANCE

- markedly reduced risk of contamination
- safety and comfort due to transparent, impermeable and tearproof bag
- suitable for all laparoscopic procedures: XL bag allows even largest specimens to be removed
- no need for an additional trocar means greater flexibility and economy
- colour-coded to facilitate choice of size

OVERALL SAFETY – EXCLUSIVE 1200 ML EXTRACTION BAG


The unique RÜSCH MemoBag – an intelligent device Teleflex provides for the extraction of specimens – will help protect against contamination in the operating field or, in case of malignant tumours, against the spreading of tumour cells.

Our well thought-out range of sizes enables you to cover all procedures, from appendectomy to nephrectomy. All three sizes of bags (from 5 cm to 13 cm in diameter) can be inserted through a 10 mm trocar. This allows for more safety, flexibility and comfort throughout the entire minimally invasive procedure.

Moreover, as the 10 mm trocar fits all sizes, our MemoBag is also very economical.

INDICATIONS FOR ENDOSCOPIC SURGERY

- colon surgery
- cystectomy
- appendectomy
- cholecystectomy
- myomectomy
- ovarian cystectomy
- lymphadenectomy
- adrenalectomy
- tubal pregnancy
- thorax surgery
- nephrectomy
- hysterectomy
- prostatectomy
- splenectomy

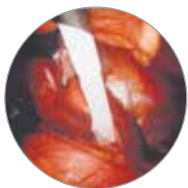

INTRODUCTION

The rolled up MemoBag opens automatically after its introduction into the operating field through a 10 mm trocar.




OPENING

After deformation, a special nitinol wire at the bags's upper rim "remembers" and adopts its original shape, thus keeping the bag open without any further manipulation. The trocar can then be used to introduce other instruments.




RESECTION

Thanks to the MemoBag's large opening, specimens can easily be placed inside the transparent, impermeable and tearproof bag. Large specimens can be cut into smaller pieces inside it, without any risk of contamination.

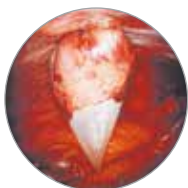




REMOVAL

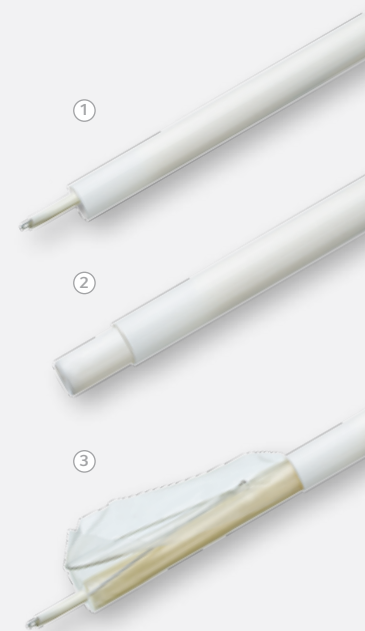The MemoBag can then be reliably closed and removed.



- ① the extraction set is introduced through the trocar
- ② the MemoBag is rolled up inside the introducer tube
- ③ the pusher releases the bag which then opens automatically

ORDERING INFORMATION

MEMOBAG				RUSCH	
	REF.	SIZE / VOLUME	Ø EXTRACTION BAG	Ø TROCAR	QTY
	332800-000010	200 ml	50 mm	10 mm	5 / box
	332801-000020	800 ml	100 mm	10 mm	5 / box
	332800-000030	1200 ml	130 mm	10 mm	5 / box

DISTRIBUTED BY:

TELEFLEX MEDICAL HEADQUARTER INTERNATIONAL, IRELAND

IDA Business & Technology Park · Dublin Road · Athlone · Co Westmeath
Tel. +353 (0)9 06 46 08 00 · Fax +353 (0)14 37 07 73 · orders.intl@teleflex.com

UNITED KINGDOM Tel. +44 (0)14 94 53 27 61 · info.uk@teleflex.com

SOUTH AFRICA Tel. +27 (0)11 807 4887 · assist.africa@teleflex.com

WWW.TELEFLEX.COM

94 05 35 · 00 00 01 · REV A · 03 13 02 · All data current at time of printing. Subject to technical changes without further notice.

teleflex